



AUTOCLAVE H12L classic

»with dry cycle and digital thermometer
»automatic pressure release

ficoinox

Dental Autoclaves & Instruments

For those who takes sterilization seriously



The **dry cycle** protects instrumental from oxidation, damage, and allows an immediate use of it.

ficoinox

Dental Autoclaves & Instruments

This autoclave incorporates a **new automatic release valve** for more comfortable use.

The **digital thermometer** allows a constant monitoring of the cycle.



New **chamber water level sensor**



Electronic detectors turn off the power in case of malfunction or improperly use.

Closed door switch insures more security for the user.

Soft touch on the control panel. **Visual and auditive signals** insure friendly operation.

A **two stage overpressure-protection system** incorporates two calibrated pressure release valves.



Thermal and electric fuses protect the device against overheating and electrical fault.

Stainless steel accessories AISI 304 L quality, suitable for Dental, General Medical, Tattooing, Veterinary and Laboratory applications.

In steam sterilization, the **AUTOCLAVE H12L classic** offers the most economical choice with high technology. It is sold in the argentine market since 1995 and sold in more than 25 countries around the world.

FEATURES	12 lts.
Diameter	220 mm
Length	300 mm
Accessories	2 trays
Voltage/ Consumption	220v — 1500watts — 7 ampers
	110v — 1000watts — 9 ampers
Sterilization selections	134°C for 3 minutes 2.1 BAR pressure
	126°C for 11 minutes 1.6 BAR pressure
	121°C for 21 minutes 1.1 BAR pressure



H12L CLASSIC
12 liters
2 trays

CERTIFICATES

Visual inspection certificate, hydraulic and performance test issued by SGS Argentina ensure the compliment of the international norm ASME 8-. The autoclave is with the international norm IEC61010-1/EN61010-1 and IEC 601-1-2 certificated-. The cycle options with the time/temperature combination are based on the International Norm EN 13060 for steam Autoclaves.

It protects your patients, your instruments and you.