

PD Temporary Cement in paste (Eugenol-Free)

Cement for temporary fillings

Composition

Zinc Oxide, Zinc Sulfate, Calcium Sulfate, excipient

Properties

PD Temporary Cement is a slightly radiopaque & antiseptic paste. Its excellent adhesion to the pulp & sealing qualities ensure a hermetic filling. The cement can be applied to the cavity whether it is either moist or dry & cause no irritation to the pulp or to the gums.

Indications

- Temporary obturation on medicaments
- Temporary filling of cavities
- Temporary setting of root posts

Directions for use

After cleaning &, where necessary, drying of the walls of the cavity, apply PD Temporary Cement with a spatula & shape to suit. The cement will dry within approximately 15 minutes after contact with the saliva. Ensure the patient does not exert pressure on the filling for at least one hour. After treatment, the cement can easily be removed with a probe.

Warning

If the jar is not securely closed after use, the top layer of the cement may harden. However, the thixotropic quality of the cement will ensure that, on mixing before use, the paste will recover its initial consistency.

Packaging

Jar of 40g

