

Oro Clean Chemie AG  
Allmendstrasse 21  
CH-8320 Fehraltorf, Switzerland  
Tel: ++41 1 954 80 30  
Fax: ++41 1 954 80 40

# Surface disinfection

## Quick disinfection

<b>PRODUCT:</b>	<b>DENTIRO<sup>®</sup> Mikro</b> Special product for the quick disinfection of medical equipment, unit surfaces and instruments (in the sense of commission directive 93/42/EEC). Can be used for other small surfaces. Aldehyde free. Easily biodegradable. Ready-to-use solution with pleasant lime fragrance.
<b>Use:</b>	Moisten the surfaces and the objects completely, allow reacting and dry slightly if necessary. Only to be used on alcohol compatible materials. With sensitive materials test compatibility before use.
<b>Activities:</b>	<input type="checkbox"/> Bactericide, fungicide within 30 seconds. <input type="checkbox"/> Tuberculocide within 30 seconds. <input type="checkbox"/> Hepatitis B <sup>1)</sup> incl. HIV within 30 seconds. <input type="checkbox"/> Hepatitis C <sup>2)</sup> within 30 seconds. <small>1) HBsAg – antigen test; 2) BVDV – surrogate virus test</small>
<b>Composition in 100 g:</b>	20 g ethanol, 28 g 1-propanol, 0.056 g quaternary ammonium compounds.
<b>Physical properties:</b>	Clear, colourless liquid with alcoholic odour and lime fragrance. Flash point: 26 °C (DIN 51 755) Density (20 °C): ~ 0.906 g/ml (g/cm <sup>3</sup> )
<b>Toxicological information:</b>	LD <sub>50</sub> - Value: > 5000 mg/kg
<b>Registrations:</b>	CE 0483 Germany: DGHM listed
<b>Shelf life:</b>	36 months (Lot and expiry-date on containment)
<b>Precautionary measures:</b>	<b>R10</b> Flammable. <b>R41</b> Risk of serious damage to eyes. <b>S2</b> Keep out of reach of children. <b>S16</b> Keep away from sources of ignition. No smoking. <b>S23</b> Do not breathe spray. <b>S25</b> Avoid contact with eyes. <b>S25</b> In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. <b>S51</b> Use only in well-ventilated areas.
<b>Test reports:</b>	Prof. Dr. med. R. Schubert, Frankfurt, D, 2001 Prof. Dr. med. H.-P. Werner, Schwerin, D, 2002, 2001 Dr. J. Steinmann, Bremen, D, 2001

These information contained herein are believed to be correct and correspond to the best knowledge available. However, Oro Clean Chemie AG, Allmendstrasse 21 CH-8320 Fehraltorf, Switzerland cannot take over the responsibility regarding its accuracy or the completeness of these information.