

Oro Clean Chemie AG
Allmendstrasse 21
CH-8320 Fehraltorf, Switzerland
Tel: ++41 01 954 80 30
Fax: ++41 01 954 80 40

Surface disinfection

Quick disinfection

PRODUCT:

DENTIRO® Wipes

Aldehyde-free cleaning and disinfecting wipes for small surfaces, instruments, medical equipment and unit-surfaces.

Use:

Cut open the refill and slide it into the container. Pull the first wipe through the opening in the lid and reset it. Moisten completely the surfaces and objects with the wipe and allow reacting. Use only with alcohol compatible materials. Close the box after each use.

Activities of the moisture solution:

- Bactericide, fungicide within 30 seconds
- Tuberculocide within 30 seconds
- Hepatitis B ¹⁾ incl. HIV en 2 minutes.
- Hepatitis C ²⁾ en 30 seconds.
- AFNOR 72170

1) HBsAg – antigen test; 2) BVDV – surrogate virus test

Composition of the moisture solution/100 g:

31,3 g ethanol, 16,7 g 2-propanol, 0,09 g quaternary ammonium compounds.

Physical properties of the moisture solution:

Clear, colourless liquid with alcoholic odour.
Flash point: 22 °C (DIN 51 755)
Density (20 °C): ~ 0.915 g/ml (g/cm³)

Toxicological information:

LD₅₀- Value: > 5000 mg/kg

Registrations:

CE 0483
Germany: DGHM listed

Shelf life:

24 months (Lot and expiry-date on container)

Precautionary measures:

R10 Flammable. **S2** Keep out of reach of children. **S16** Keep away from sources of ignition - No smoking. **S24/25** Avoid contact with skin and eyes. Keep in a cool and dry place.

Test reports:

Dr. J. Steinmann, Bremen, D, 2001, 2002
Dr. A. Sorger, Bischofshofen, 2001
Prof. Dr. med. H.-P. Werner, Schwerin, D, 2001
LHM, P. Rösli, Mülligen, CH, 2001

These information contained herein are believed to be correct and correspond to the best knowledge available. However, Oro Clean Chemie AG, Allmendstrasse 21 CH-8320 Fehraltorf, Switzerland cannot take over the responsibility regarding its accuracy or the completeness of these information.