



Impression materials are the starting point of many dental operations. High-quality impressions give the dentist confidence in a good end result. Cavex offers a well-tried and reliable range of alginates, hydrocolloid and auxiliary materials developed according to the requirements of the dental practice. For dental use only.

Cavex ColorChange, the alginate with virtual intelligence.

Cavex ColorChange 'knows' when the mixing time has ended and the alginate paste can be placed in the tray: the colour will change from lilac to pink. Subsequently the colourchange alginate will indicate exactly when the impression has hardened and the tray can be removed from the mouth: then you'll see the colour change again, from pink to white. The strong but elastic impression remains dimensional stable for at least 5 days in an ordinary sealed plastic bag to enable one or even two gypsum models to be made that will be accurate right down to the smallest details.





CAVEX

Cavex Holland BV
Harmenjansweg 19
P.O. Box 852
2003 RW Haarlem
The Netherlands

T +31 23 530 77 00
F +31 23 535 64 82
dental@cavex.nl
www.cavex.nl

Colourchange alginate, an accurate procedure for an accurate result.

- **innovative reaction mechanism**

For more than 50 years Cavex has been producing and perfecting alginate products which are now used worldwide in dental practice. Cavex ColorChange is the youngest generation of these products, in which the proven qualities of Cavex alginates are optimised and combined with a built-in colour indicator for an exact timing during the procedure.

- **5 days dimensional stability**

Cavex ColorChange impressions meet the highest standards of accuracy (ISO 1563/ADA 18). They are elastic, tearproof and almost free from shrinkage.

- **rapid, complete hardening reaction**

You can store them for at least 5 days in an ordinary sealed plastic bag. The compatibility with gypsum is optimum. You can pour out immediately, later or even twice - whatever suits you or your technician best.

- **highly elastic, tearproof**

Patient-friendly and practical for the dentist.

Even patients seemed to be fascinated by the colour change. A little piece of mixed alginate changing colour in the hand keeps the patient diverted and amused during the impression-taking, so you'll get a better impression. And, of course, the short hardening time (1 minute) and the pleasant flavour of Cavex ColorChange also help to keep the patient happy.

- **smooth surface, optimum gypsum compatibility**

For the dentist the satisfaction lies in the ease of the dust-free dosing and mixing and the rapid, complete moisture absorption, which gives a smooth, homogeneous mixture without clumping. The alginate mass flows under pressure and is stable on the tray. The impression is elastic and easy to remove from the mouth, even in the case of undercuts. Extremely suitable for orthodontics.

- **dust-free dosing**

Cavex ColorChange is available from your dental supplier. For further information about product composition and applications, please contact Cavex, +31 23 530 77 00 - Dental Customer Service or surf to www.cavex.nl

Beginning of the mixing process of water and alginate powder.

End of mixing process, tray can be filled and then placed in the patient's mouth.

End of hardening, the tray can be removed.



CAVEX FOR DENTAL USE ONLY

ZYEC2000/5-0