

CAVEX

H



Hygiene is a precondition for a good dental practice. To help you maintain it, Cavex supplies efficient and effective products which are not harmful to man or the environment. Products that are carefully matched to the requirements of dentists, specialists, dental assistants, dental nurses and dental technicians. For dental use only.

At last: the intelligent solution for removing alginate residues quickly and easily - Cavex GreenClean.

After all, who can make a better alginate solvent than the people who make alginate itself? Cavex GreenClean is proof of that. The most effective way to remove alginate and plaster residues is by dissolving them, not by scraping or scouring. Get impression trays, spatulas and other instruments spotlessly clean again quickly and easily.

Cavex GreenClean is a gentle, user-friendly product that spares both your instruments and your skin. That's what you call an intelligent solution!



Cavex Holland BV
P.O. Box 852
2003 RW Haarlem
The Netherlands

H

CAVEX

Cavex Holland BV
Harmenjansweg 19
P.O. Box 852
2003 RW Haarlem
The Netherlands

T +31 23 530 77 00
F +31 23 535 64 82
dental@cavex.nl
www.cavex.nl

The secret of Cavex GreenClean: the breaking down of the organic matrix.

- **pH-neutral**

As is often the case, the best way of removing really stubborn alginate and plaster residues is more obvious than you might think. All we need to do is to break down the organic matrix and the entire residue will then just fall apart. Rinse with water, or just do a bit of brushing, and a clean, gleaming and spotless impression tray is again ready for use.

- **kind to the skin**

It works just as effectively and easily with spatulas and other instruments which come into contact with alginate.

- **biodegradable**

Add two scoops of Cavex GreenClean to half a litre of water. Remove any loose alginate residues. Immerse the instruments for 15-30 minutes. Then rinse or brush off. It's done! No more scraping, no more scouring. And if you use warm water or an ultrasonic bath then things will work even faster.

- **1 kg > 500 clean trays**

- **fresh aroma**

pH-Neutral, biodegradable. That's good for materials, for man and the environment.

Cavex GreenClean only goes to work on the alginate or plaster residues, and not on the trays or spatulas themselves, which remain spotless and untarnished. And, just as importantly, it doesn't go to work on the skin either! Cavex GreenClean is also pH-neutral and biodegradable, so residues can be removed in the sink without problems. The only thing that's left behind is a fresh, light citrus aroma.

The transparent pot with measuring scoop contains 1 kg of Cavex GreenClean, enough to clean more than 500 trays. Not only effective, but economical too!

Cavex GreenClean is available from your local dental supplier. For further information on product composition and uses, please contact Cavex, +31 23 530 77 00 - Dental Customer Service.



CAVEX FOR DENTAL USE ONLY

ZYEC2000/1-0