

## PD Zircon-Fluor Treatment Prophylaxis Paste

---

### Zirconium Silicate Dental Treatment Paste

#### Composition

Stannous Fluoride 9%, Zirconium Silicate, Mint Oil, excipient.

#### Properties

PD Zircon-Fluor Dental Treatment Paste contains Zirconium Silicate, an abrasive which clean & polishes the teeth with a minimum of abrasion risk for the tooth surface & 9% Stannous Fluoride, an excellent agent for topical application of Fluor for the prevention of tooth decay.

#### Directions for use

After a thorough scaling, use the preparation with PD RUBBER POLISHING CUPS at low speed, treating the tooth wit fresh paste for about 10 seconds. Then reapply a sufficient quantity of the paste to keep the teeth moist, for about 4 minutes.

#### Cautions

- Staining or pigmentation has been reported as resulting from the topical application of concentrated Stannous Fluoride solutions on teeth with carious lesions, hypo-calcified areas in the enamel or at the margin of Silicate restorations, particularly in the case of patients with oral hygiene.

- As the Stannous Fluoride has a strong astringent effect on the tissues, which could be painful, care should be taken to avoid any contact of the paste with the gum. Isolation of the lips with cotton rolls & the use of rubber dam are therefore recommended.

#### Packaging

Jar of 45g

