

PD Creosote

Disinfectant & analgesic

Composition

Beechwood Creosote

Properties

Creosote is composed chiefly of Guaiacol & Cresol & is used for its antiseptic & analgesic actions. It is applied topically, preferably in diluted solutions either in alcohol or in an essential oil.

Indications

- Disinfection of the root-canal
- Treatment of putrescent infections, stomatitis
- Alleviation of post-operative pain after extraction

Direction for use

- Use a plastic pipette or a cotton pellet to apply a droplet of the solution to the area to be treated

Warning

Do not swallow. Avoid contact with skin or oral mucous. Where necessary, rinse abundantly with water.

Storage

Store at room temperature between 5°C & 30°C, protected from direct light.

Packaging

Bottle of 15ml

